

# Etac Socky stocking aid

## Etac Butler buttoners

To reach down to the feet or to fasten small buttons or zippers can be distressing when pain restricts the mobility or fine motor skills are reduced. These problems can be complex and one model may not be appropriate for everyone. Here is a choice of three stocking aids and two buttoners.



*The plastic hooks under the nylon fabric holds the sock in place and makes it easy to apply the sock.*

### Etac Socky stocking aid, short

Socky short is the perfect stocking aid for both short and knee long socks including flight socks. It consists of a plastic base with two hooks under a pre folded nylon fabric.

#### How to use Etac Socky Short

The sock is simply pulled onto the Socky fabric until the top of the sock reaches over the hooks. Place the string behind the calf and pull. Socky short slides up around your foot and lower leg with ease, placing the socks perfectly.



*The handle makes Socky Long easy to use when the strength and mobility are restricted in hands and arms.*

### Etac Socky stocking aid, Long

Socky long is the ideal solution when hand strength and mobility is restricted or you need to reduce the friction between the sock and floor/carpet. It can be used for short as well as knee long socks and can also be used for tights. Socky long consists of a long plastic sheet placed under a pre folded nylon fabric with a handle on top.

#### How to use Etac Socky Long

The sock is simply placed around the Socky fabric and pulled to the top. The handle is used to pull Socky long up, while the plastic sheet slides on the floor and ensures that the sock stays in place. For tights, use two Socky longs connected with Etac Socky linking device.



*This Socky model also facilitates for a care person to assist when putting on support or compression stockings.*

## Etac Socky stocking aid for support and compression stockings


This stocking aid is designed to suit different lengths and types of support and compression stockings. It consists of two long nylon fabric sections that connect with Velcro. Each part has a large handle to grip and to pull the stocking over the foot and leg.

### How to use Socky Long

Place the support stocking onto the Socky. The fabric parts divide when you pull up the stocking. It also facilitates for a care person to assist.

## Etac Butler buttoners

The Butler buttoners are available in two models. The model with a round grip can be used for both big and small buttons and also zippers. The model with a long handle can be used for all types of buttons, even jeans metal type buttons.

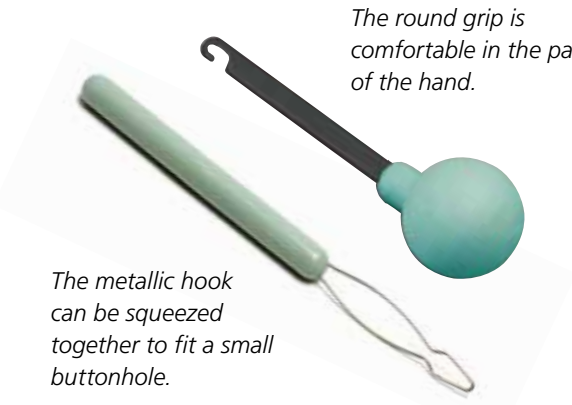

*The round grip is comfortable in the palm of the hand.*

*The metallic hook can be squeezed together to fit a small buttonhole.*



**Instruction video at [etac.com](http://etac.com)**  
How to use Etac Socky



Description	Item no.
Socky stocking aid, short	80601002-2
Socky stocking aid, long	80601001-2
Socky Linking device	80601021
Socky stocking aid for support and compression stockings	80601003-2
Butler buttoner, round	80602001
Butler buttoner, long	80602004

### Weight

Socky short: 94 g (3.3)  
 Socky long: 146 g (5.1 oz)  
 Socky stocking aid for support and compression stockings: 90 g (3.2 oz)  
 Butler, round: 25 g (0.9 oz)  
 Butler, long: 18 g (0.6 oz)

### Material

Socky stocking aids:  
 Fabric: Polyamide  
 Plastic parts: Polypropylene  
 Butler buttoners:  
 Handle: Polystyrene  
 Butler round, shank: Epoxy coated metal  
 Butler long, hook: Galvanised wire

### Design

HM Center

### Length

Socky short, 40 cm (15¾")  
 Socky long, 60 cm (23½")  
 Socky stocking aid for support and compression stockings: 100 cm (39¼")  
 Butler, round: 13 cm (5")  
 Butler, long: 22 cm (8¾")

### Patent

Socky short and Socky long is patented.