
# Etac Kaskad

Manual


etac®

78630.7 23-01-17

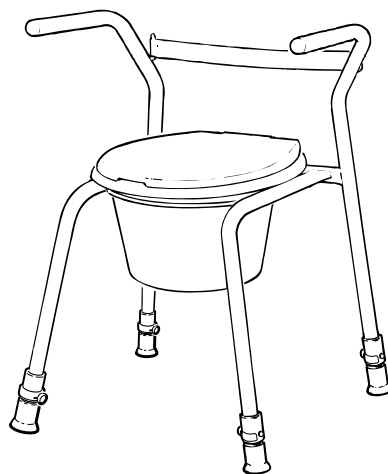sv no da en de fi fr nl it es pt

A

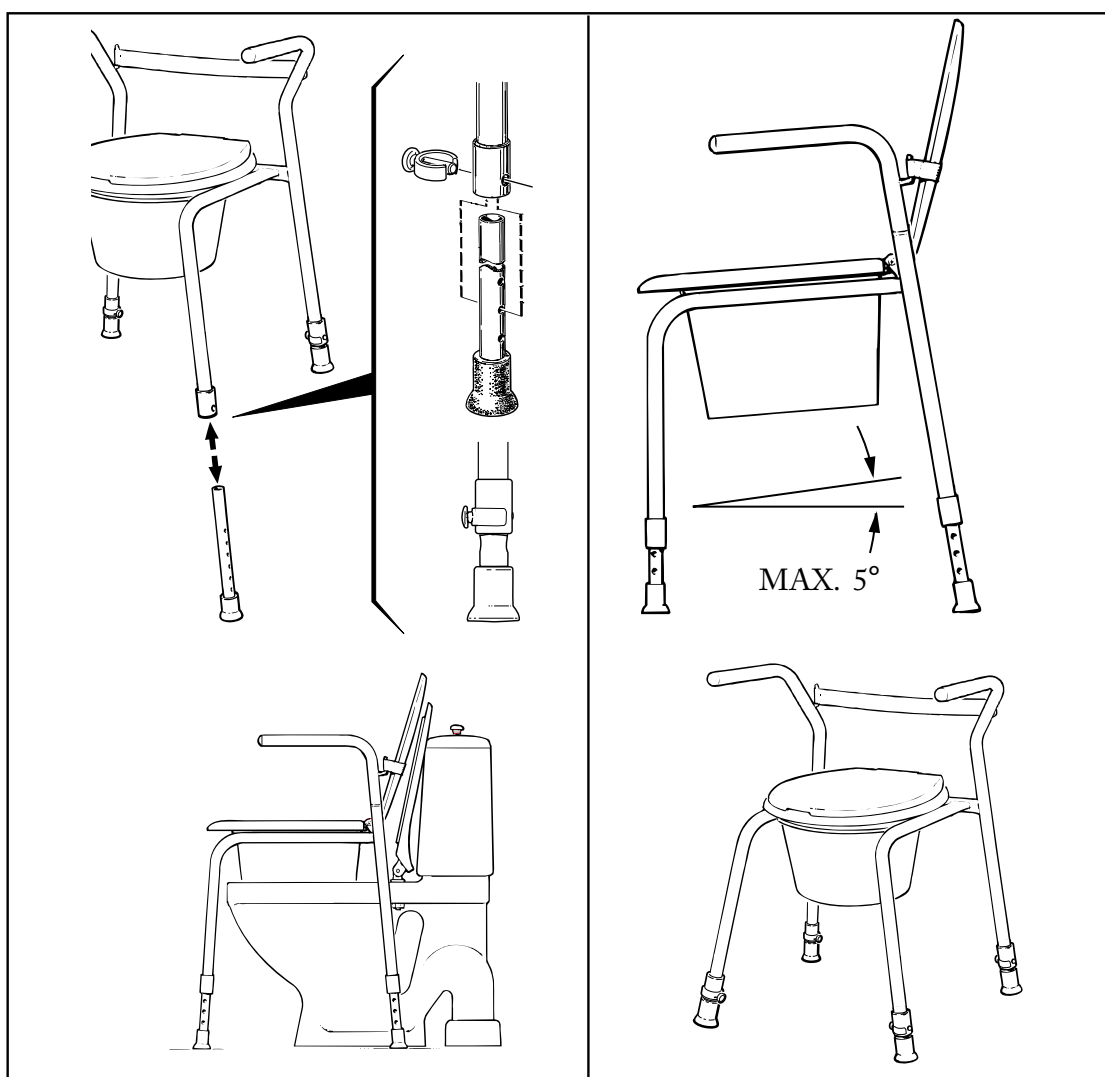## B

Max  
150 kg  
330 lbs

## C

25 cm	53 cm	58 cm	70-82 cm	46-49 cm	22 x 29 cm	45-57 cm	5,2 kg

sv no da en de fi fr nl it es pt

sv

## Svenska

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**Tekniske data** ..... *Figur C*  
**Tilbehør:**

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**en** English

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at [www.etac.com](http://www.etac.com). You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

**Intended use**

The Etac Kaskad free-standing commode is designed for use as a room toilet or as a free-standing raiser over a toilet. It has been developed to facilitate toilet use by persons with restricted mobility. The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

**Contra indications**

There are no known contraindications.

**Installation and use** ..... *Figure A***Maintenance and safety** ..... *Figure B*

**Warning:** A defective product must not be used.

**Warning:** A height difference of more than one hole is not allowed for chair legs.

**Warning:** When moving sideways, do not lean on just one armrest.

**Max user weight:** 150 kg.

**Cleaning:** Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

**Decontamination:** Can be decontaminated at max. 85°C/185°F for 3 minutes if required.

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**Guarantee, service life**

5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](http://www.etac.com).

Service life 10 years. For complete information regarding the service life of the product, see [www.etac.com](http://www.etac.com).

The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

**Storage**

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four months), its function must be checked by an expert before use.

**Technical data** ..... *Figure C***Accessories:**

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Etac Supply Center AB