

Patient Handling Low Friction Gloves

OPERATION AND SAFETY INSTRUCTIONS

BetterLiving Patient Handling Slide Gloves are made of silicone coated PU, featuring a non-slip interior and a low friction silicone exterior.



 Please read before use

Usage

The Low Friction Glove can be used in minor manual handling activities to manipulate and adjust a user's position when low friction movements are required. The Low Friction gloves are worn on the hands of the Carer. Gloves can be worn on both hands or on a single hand as required. This allows flexible patient handling for movements like adjusting the hips, shoulders and extremities.

They are designed to assist in positioning of patients on slide sheets, reducing friction between the sheet and the caregiver's hands for easier maneuvering.

These gloves also serve as a dressing aid, reducing friction between the skin and clothing. They are used with hoists or lifting aids to reduce shearing and facilitate movements. Occupational Therapists may find them useful for seating patients on cushions, providing control over bony areas.

Regular Maintenance

Check gloves frequently for any signs of damage, tearing or fraying. If in doubt, discontinue use. Condition of the gloves may deteriorate over time, due to prolonged exposure to sunlight, the cumulative effects of laundering, contact with sharp or abrasive surfaces or edges, etc.

The product must therefore be inspected for signs of damage or wear prior to each use. In the event that signs of damage or wear are apparent, the product should be replaced.

Care and cleaning

The product should be cleaned regularly and should not be used across multiple users without appropriate cleaning and infection control consideration. If the product becomes soiled or increased infection control practices are required, use a mild soap solution and rinse carefully. Do not use harsh detergents, solvents or bleach. For increased infection control, machine wash up to 80°C, increasing and decreasing the temperature in stages to avoid damage and shrinkage. Air dry when possible, using a cool tumble dryer temperature not exceeding 50°C if necessary. Never place in a hot drum or expose to extreme temperatures.



Specification Table

PH-SS-G	
COLOUR	Blue
WASHABLE	Yes
RE-USABLE	Yes
MATERIAL	Silicone Coated Polyurethane (PU)
HEMMED	Yes
DIMENSIONS	50 x 20 cm
WARRANTY	1 year

WARRANTY STATEMENT

Limited Warranty

This warranty is provided by Novis Healthcare (ABN 45 102 735 491) of Unit 12, 12 Mars Road, Lane Cove NSW 2066

Novis Healthcare (Novis) products are manufactured to the highest quality standards and are thoroughly tested and inspected before leaving our factory. In addition to any statutory rights and remedies you may have, Novis warrants all of its products sold directly or via an Authorised Novis Australia Dealer against defective workmanship and faulty materials from the date of purchase by the end user for a period of twelve months unless otherwise specified for that product and its components.

Warranty Claims

To claim under this warranty, please contact Novis Healthcare and have your receipt or proof of purchase available. Novis Healthcare may need to assess the defect before determining any claim, and additional information may be requested to process your claim. Claims without proof of purchase may not be able to be processed.

Novis Healthcare may at its option inspect the goods on site or require them to be returned to its premises or one of its Authorised Service Agents in person or freight prepaid by you.

Novis will undertake at its option, to repair or replace, free of charge, each product or part thereof on the condition that:

- ~ The product found on examination, to be suffering from a manufacturing defect;
- ~ The product or relevant part has been serviced regularly by Novis or one of its Authorised Service Agents and has not been subjected to misuse, neglect or been involved in an accident;
- ~ The repairs are not required as part of normal wear and tear.

At our option

- ~ Goods repaired may be replaced by refurbished good of the same type rather than being repaired.
- ~ Refurbished parts may be used to repair goods.

Novis Healthcare will not be held responsible for any repair other than those carried out by it or one of its Authorised Service Agents.

Warranty repairs do not extend the length of the warranty period.

Limited Liabilities

Our liability under this manufacturer's warranty is subject to us being satisfied that a defect was caused by faulty parts, manufacture or workmanship, and was not caused or substantially contributed to by other factors or circumstances beyond our control, including (but not limited to) defective installation, maintenance or repair, product modification or alteration, any neglect, misuse, or excessive use, normal wear and tear or failure to follow manufacturer's instructions.

Important notice for Australian consumers

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law.

You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

To obtain compensation, you will need to provide documentary evidence of the loss or damage suffered and documentary evidence that such loss or damage was a reasonable foreseeable consequence of a failure. Novis Healthcare to comply with a consumer guarantee under the Australian Consumer Law. Subject to the provisions of the Australian Consumer Law, Novis Healthcare excludes, to the fullest extent permitted by law, all liability in respect of loss of profit or other economic loss, direct to indirect or consequential, special, general or other damages or other expenses or costs which may include negligence.